# markennovy

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USER GUIDE FOR SOFT CONTACT LENSES  
(Also available at [www.markennovy.com](http://www.markennovy.com))

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	Caution
	Consult instructions for use
	Sterilization method- Moist heat steam
	Manufacturer
	Manufacture date
	Lot number
	Use-by date
	Recycleable
	Keep away from sunlight
	Do not use if package is damaged
	Do not use tap water to clean your lenses or storage case
	Do not re-use

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Referencias.

1.Sankaridurg P. Novel contact lenses designed to slow down progression of myopia: 2 years results (CRTc2013\_015 Internal Report), 2018.

## ES

## FOR SOFT CONTACT LENSES

### INTENDED USE:

All mark'envoy contact lenses are intended for the optical compensation of myopia (short sightedness) and hyperopia (also called Hypermetropia, long sightedness) and may include astigmatic or presbyopic variations.

Additionally, MYLO contact lenses correct myopia and may reduce the rate of myopia progression. Reduction of myopia progression was observed in a clinical study in children (7 - 13 yrs!) that were asked to wear them all waking hours. Results showed more myopia progression reduction in the ones that wore them 6 and 7 days per week. Based on this our recommendation is to wear the lenses at least 6 days per week and for all waking hours.

The fitting of contact lenses has to be done by a qualified eyecare practitioner (Ophthalmologist, Optometrist or Optician) and the user needs to have healthy eyes.

mark'envoy offers a wide range of contact lenses manufactured from various materials water content and geometries: Spheric, Aspheric, Toric, Multifocal, and Multifocal Toric.

Type of material	Classification	Brands
Silicone hydrogel 75%	Filcon 5B (60) [75%]	Blugen, Blussentials, Blu:kidz,* Saphir, Saphir RX, Brilliant, MYLO, EDOF
Origen Technology 80%	Filcon 2 (60) [80%]	Gentle 80
Origen Technology 59%	Filcon 2 (30) [59%]	Gentle 59
GMA 59%	Filcon 2 (24) [59%]	Equilibria, 5T, SPH 5
GMA 49%	Filcon 1 (15) [49%]	Quattro
2-Hema 38%	Filcon 1 (10) [38%]	ET43, ES 43
Methafilcon 55%	Filcon 4 (19) [55%]	Xtensa, Xtensa RX, Seven, Seven RX, Versa
Filcon IV 1 52%	Filcon 4 (18) [52%]	IndiGO, Jade*

\*IndiGO, Jade, Blugen, Blu:kidz & Blussentials have a UV filter Class I per ISO 18369-2:2017 (>90% UVA & > 99% UVB) and blue blocking light filter.

This guide includes information and instructions related to handling, maintenance, precautions, and contraindications to ensure safe and efficient use of your soft contact lenses. With this guide's help, and your practitioner's advice, you will be able to enjoy the advantages of wearing contact lenses while achieving optimum health for your eyes.

### PLEASE READ THIS GUIDE CAREFULLY AND SAVE IT FOR FUTURE REFERENCE

mark'envoy lenses are delivered safely in buffered saline solution and are individually prescribed for daily use. The lenses are sterilized via moist heat and remain sterile until the seal has been broken. Should you receive a vial or blister with its original seal damaged or opened, do not use the lenses and consult your practitioner.

When properly fitted, all mark'envoy contact lenses are suitable for day and night driving. The CE mark and the number of the Notified Body confirm that the product complies with the requirements of the European Medical Device Directive. Packaging labels specify the expiration month and year, lot, and see instructions.

#### 1. BEFORE HANDLING THE LENSES

- 1.1. Wash your hands with a neutral soap, and then rinse and dry them thoroughly.
- 1.2. Gently shake the contact lens blister or vial before opening it.
- 1.3. Do not handle the lenses after applying creams, oil, or body lotions to your fingers.
- 1.4. It is recommended to keep your fingernails short and well-trimmed to avoid damaging your lenses.
- 1.5. Become accustomed to always handling one lens before the other to avoid mixing up the correct lens for each of your eyes.
- 1.6. Blister: Pull off the blister foil and spill the content on the palm of your hand. If necessary, slide the lens out of the blister using your fingertip.
- 1.7. Vial: unscrew the top and spill its contents onto the palm of your hand. If the lens stays inside the vial, refill it with multipurpose or saline solution, close the vial, gently shake it and spill the content again on the palm of your hand. Repeat the operation if necessary.

#### 2. INSERTING YOUR LENSES

##### RIGHT LENS:

1. Check to make sure the lens is not turned inside out by method (a) or (b).
- a) Take the lens from the palm of your hand and gently pinch it with your thumb and index finger to fold it in half. If the edges touch one another, the lens is in the correct position. If the edges do not touch one another, the lens must be turned inside out before placing it in the eye.
- b) Observe whether the edge of the lens is correct (Fig. 1) or incorrect. (Fig. 2)
2. Rinse the lens with saline solution as recommended by your eye care practitioner.
3. Place the right lens on the index finger of your right hand.
4. With your left hand pull up the upper eyelid just under the eyelashes.
5. With your right middle finger, pull down the lower eyelid, look up and place the lens gently on the lower white part of your eye. The lens will position itself (Fig. 3).
6. Look down, slowly release your eyelid, and blink.
7. To help the lens centre itself, close your eye and gently rub your eyelid in a circular motion.

##### LEFT LENS:

After putting in the lenses, they should be comfortable and give you clear vision. Any discomfort experienced could be caused by a foreign body, which has been trapped between the lens and your eye whilst you were putting in the lens. Unclear vision could be caused by mixing up of the right and left lenses, or the lens not being correctly centred on the cornea.

#### 3. REMOVING YOUR LENSES

##### RIGHT LENS:

1. Make sure your hands are completely dry.
2. Make sure the lens is centred on the cornea. This can be done by looking in the mirror or covering your other eye. If your vision is clear, the lens is centred correctly.
3. Pull down the lower eyelid with your right middle finger, look upwards, and place your index finger on the lens.
4. Slide the lens downwards without raising the index finger and softly pinch the lens using your finger and thumb.
5. Rinse the lens with saline solution, never your fingernails. (Fig. 4).
5. If the lens cannot be removed, blink a few times and/or apply a lubricating eye drop and then try again following the previous steps.
6. Clean and rub the lens, then disinfect, using the recommendations of your practitioner. (Fig. 5)

##### LEFT LENS:

Repeat the same steps followed for the right eye.

#### 4. MAINTENANCE

DAILY DISINFECTION: "All in one" or peroxide solutions for soft contact lenses can be used. The solution must be fresh and replaced everyday.

If contact lenses are not worn for several days, they must be disinfected before use.

DAILY MAINTENANCE: Rub and use surface cleaner for soft contact lens deposits.

ELIMINATION OF PROTEIN DEPOSITS: Use a protein removal product, or an "all in one" solution containing a protein removal agent.

It is recommended that the lens storage case is changed every 90 days. Tap water, saliva, or products intended for gas permeable contact lenses should not be used to rinse or clean soft contact lenses.

Always adhere to expiration dates of contact lens care solutions and adhere to the recommendations in the manufacturer's product information.

To rinse the case of contact lenses with multipurpose solution and dry at air, at least one time per week. Proper contact lens care greatly reduces the risk of all contact lens-related eye infections, including those caused by Acanthamoeba.

#### 5. CONTRAINDICATIONS

- Corneal hyposthesia
- Eye or eyelid inflammation or infection
- Induced or exacerbated allergic reactions due to CL wear or CL solutions use.
- Any eye disease, injury or abnormality that affects the cornea, conjunctiva or eyelids (recurrent corneal erosions, epithelial fragility, severe dry eye, red or irritated eyes, etc.)
- Any previously diagnosed condition that makes CL wear uncomfortable
- Any systemic disease that may affect the eye or be exacerbated by wearing CLs.

#### 6. PRECAUTIONS

1. Do not use the lenses if the original seal is broken.
2. Always disinfect the lenses before use.
3. Do not change the maintenance system, lens type, or lens parameters without first consulting your practitioner.
4. If a lens accidentally dries out, dispose of it.
5. Eyes should always be closed when applying hair spray as it may damage your lenses.
6. Make-up can discolour the lenses, so it is recommended to put your lenses in before applying make-up and remove lenses before taking off make-up.
7. Lenses should not be used in swimming pools, lakes and beaches except with goggles. If for any reason lenses come into direct contact with the water, wait 30 min before removing them and disinfect them immediately. Contact with unsterile water (e.g. from the tap, bottled, bath, pool, or water sports) increases the risk of microbial infection.
8. Do not use lenses in environments exposed to harmful or irritating vapours or liquids.
9. Consult your eye care practitioner about using lenses while performing activities that could be considered contact sports.
10. Avoid any contact between the lenses and soap or shampoo, as the eyes could become irritated.
11. Adhere to the regular eye check-ups recommended by your practitioner.
12. To guarantee optimum health for your eyes, renew your lenses regularly in accordance with the instructions included in the packaging or the wearing scheduled stipulated by your practitioner.
13. The lenses should not come into contact with medications or other substances that could discolour them or affect your health. Carefully read the instructions for use or consult a pharmacist or your eye care practitioner.
14. Avoid extreme temperatures: do not store inside a refrigerator or freezer, in glove compartment of a car during warm days, in direct sunlight or near heat sources.
15. Although some lenses provide UV and blue light protection, they are not sunscreen substitutes for sunglasses.
16. Do not share lenses with anyone.
17. Do not wear lenses while sleeping.
18. To renew the solution of case each time that you wear contact lenses to ensure a correct disinfection.
19. Always keep spare lenses or back-up spectacles with you.

#### 7. UNDESIRABLE SIDE-EFFECTS

In general, the potential undesirable side-effects when fitting contact lenses may happen mainly on the cornea and the conjunctiva that appear as a fitting complication:

1. Inflammation
4. Mechanical issues
2. Infection
5. Allergy exacerbated by contact lens wear
3. Hypoxia
6. Dryness

The worst scenarios for reportable serious incidents are: permanent deterioration of vision (loss of structure or function) and/or damage in eye that requires a medical treatment which leaves sequel. In the event of a serious incident relating to the product, the eye care professional should be informed, who will then be responsible for communicating the issue to the manufacturer and, if needed, to the competent health authorities.

#### 8. WARNING

Do not use your lenses if they are damaged or their expiry date exceeded.

Contact lens cases can be a source of bacterial growth.

Not following the wearing guidelines given by the eye care professional can lead to loss of vision and serious injury to the eyes.

The risk of ulcerative keratitis among contact lens users who smoke is greater than among non-smokers. Certain medications may cause blurred vision, lens dryness, or lens, discomfort (antihistamines, decongestants, diuretics, muscle relaxants, oral contraceptives, tranquilizers, and motion sickness medications.) if chemicals of any kind (household products, gardening solutions, laboratory chemicals, etc.) are splashed into the eyes, FLUSH THE EYES IMMEDIATELY WITH TAP WATER AND IMMEDIATELY CONTACT THE EYE CARE PRACTITIONER. If you experience any of the following symptoms: Irritation, discomfort, pain, red eyes, foreign body sensation, excessive tearing, abnormal secretions, major changes in vision or haloes around lights, REMOVE YOUR LENSES IMMEDIATELY AND CONSULT YOUR EYE CARE PRACTITIONER. If you use lenses while pregnant, you may experience symptoms of dryness and slight changes in refraction. Consult your vision specialist if this happens. Lenses can be used by disabled, hearing or visually impaired patients. Any special training needed to insert, remove or care for their contact lenses) will be provided by the eye care practitioner, who is responsible for determining a patient's capacity for contact lens wear. Children (to 6 years) will require an adult to insert, remove and care for their lenses.

#### 9. WEARING SCHEDULED RECOMMENDED BY YOUR PRACTITIONER

DAILY WEAR (HOURS):

REPLACEMENT: □ Annual □ 3-Monthly □ Monthly □ Weekly □ Daily\*

(\* Does not require maintenance (Versa One Day Toric & IndiGO).

#### 10. CONTACT LENS DISPOSAL

Contact lenses shall be disposed of in waste bins after use and never be disposed of in drains or toilets.

Referencias.

1.Sankaridurg P. Novel contact lenses designed to slow down progression of myopia: 2 years results (CRTc2013\_015 Internal Report), 2018.

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