GU 400554 v19
 Texto revisado 2024-07-30
 Text revised 2024-07-30

 mark'ennovy Personalized Care S.L.
 Avenida Ada Lovelace, 12
 28906 Getafe (Madrid) - SPAIN
www.markennovy.com

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Referencias.

1.Sankaridurg, P. et al. Myopia control with novel central and peripheral plus contact lenses and extended depth of focus contact lenses: 2 year results from a randomised clinical trial. *Ophthalmic Physiol. Opt.* 39, 294–307 (2019).

FIG.5

INTENDEDO USE: Daily use soft corrective contact lenses to compensate myopia and hyperopia combined or not with astigmatism and/or presbyopia. Additionally, MYLO contact lenses may reduce the rate of myopia progression in simple myopia or where astigmatism is present with myopia. Reduction of myopia progression was observed in a clinical study in children (aged 7 to 13 years*) that were asked to wear contact lenses with MYLO's optical design during all waking hours. Results showed more myopia progression reduction in the ones that wore them 6 and 7 days per week. Based on this, our recommendation is to wear the lenses at least 6 days per week and for all waking hours.

The fitting of contact lenses must be done by a qualified eyecare practitioner (Ophthalmologist, Optometrist or Optician) and the user must have healthy eyes.

PRODUCTS: Mark'ennovy offers a wide range of contact lenses manufactured from various materials and geometries: spheric, toric, multifocal and multifocal toric.

Type of Material	Classification	Brands
Silicone hydrogel 75%	Filcon 5B (60) [75%]	(Saphir RX, Brilliant, MYLO, EDOF, George, Aera RX)** (Blu:gen, Blu:essentials, Blu:kidz)** Saphir 3-Monthly
Origen Technology 80%	Filcon 2 (60) [80%]	Gentle 80
Origen Technology 59%	Filcon 2 (30) [59%]	Gentle 59, Yvonne, EDOF 59
GMA 59%	Filcon 2 (24) [59%]	Equilibria, 5T, SPH 5
GMA 49%	Filcon 1 (15) [49%]	Quattro
Methafilcon 55%	Filcon 4 (19) [55%]	Xtensa RX, Seven RX
Silicone hydrogel 45%	Filcon 5C (70) [45%] c	(Xtensa SiHy, Aera, Insight)*

*Saphir RX, Brilliant, MYLO, EDOF, Xtensa SiHy, George, Aera, Aera RX and Insight have a class 1 UV filter per ISO 18369-2:2017 (>90% UVA & >99% UVB); **Blu:gen, Blu:essentials and Blu:kidz have also a blue light blocking filter.

This guide includes information and instructions related to handling, maintenance, precautions, and contraindications to ensure safe and efficient use of your soft contact lenses. With this guide's help, and your practitioner's advice, you will be able to fully enjoy the advantages of wearing contact lenses while achieving your best results for your eyes.

PLEASE READ THIS GUIDE CAREFULLY AND SAVE IT FOR FUTURE REFERENCE

Mark'ennovy lenses are delivered sealed in buffered saline solution and are individually prescribed for daily use. The lenses are sterilized via moist heat and remain sterile until the seal has been broken. Should you receive a vial or blister with its original seal damaged or opened, do not use the lenses and consult your practitioner.

Proper contact lens use and storage instructions are suitable for day and night driving. The CE mark and the number of the Notified Body confirm that the product complies with the requirements of the Directive 93/42/CEE and 2007/47/CE and transition to the Medical Device Regulation (EU) MDR 2017/745. Packaging labels specify the expiration month and year, lot, and where to see instructions.

- 1. BEFORE HANDLING THE LENSES**
- 1.1. Wash your hands with a neutral soap, rinse and dry them thoroughly.
- 1.2. Gently shake the contact lens blister or vial before opening it.
- 1.3. Do not handle the lenses after applying oil, or body lotions to your fingers.
- 1.4. It is recommended to keep your fingernails short and well-trimmed to avoid damaging your lenses.
- 1.5. Become accustomed to always handling one lens before the other to avoid mixing up the correct lens for each eye.
- 1.6. Blister: Pull off the blister foil and empty the content on the palm of your hand. If necessary, slide the lens out of the blister using your fingertip.
- 1.7. Vial: unscrew the top and empty its content onto the palm of your hand. If the lens stays inside the vial, refill it with multipurpose or saline solution, close the vial, gently shake it and spill the content again on the palm of your hand.
- 2. INSERTING YOUR LENSES**
- RIGHT LENS:**
1. Check to make sure the lens is not turned inside out by method (a) or (b).
- a) Take the lens from the palm of your hand and gently pinch it with your thumb and index finger to fold it in half. If the lens does not touch one another, the lens is in the correct position. If the edges do not touch one another, the lens is in the correct position. If the edges do not touch one another, the lens must be turned inside out before placing it in the eye.
- b) Observe whether the edge of the lens is correct (Fig. 1) or incorrect (Fig. 2).
2. Rinse the lens with multipurpose or saline solution as recommended by your eye care practitioner.
3. Pull down the lower eyelid with your right middle finger on the index finger of your right hand.
4. With your left hand pull up the upper eyelid just under the eyelashes.
5. With your right middle finger, pull down the lower eyelid, look up and place the lens gently on the lower white part of your eye. The lens will position itself (Fig. 3).
6. Look down, slowly release your eyelid, and blink.
7. Clean and rub the lens, then disinfect, using the recommendations of your practitioner (Fig. 5).
- LEFT LENS:** Follow the same steps as for the right eye.
- After putting in the lenses, they should be comfortable and give you clear vision. Any discomfort experienced could be caused by a foreign body, which has been trapped between the lens and your eye whilst you were putting in the lens. Unclear vision could be caused by mixing up of the right and left lenses, or the lens not being correctly centred on the cornea.
- 3. REMOVING YOUR LENSES**
- RIGHT LENS:**
1. Make sure the lens is centred on the cornea. This can be done by looking in the mirror or covering your other eye. If your vision is clear, the lens is centred correctly.
2. Breathe out and use your index and middle finger to push upwards, and place your index finger on the lens.
3. Slide the lens downwards without raising the index finger and softly pinch the lens using your finger and thumb. Always use your fingertips, never your fingernails (Fig. 4).
4. If the lens cannot be removed, blink a few times and/or apply a lubricating eye drop and then try again following the previous steps.
5. Clean and rub the lens, then disinfect, using the recommendations of your practitioner (Fig. 5).
- LEFT LENS:** Repeat the same steps followed for the right eye.
- 4. MAINTENANCE**
- DAILY DISINFECTION:** Multipurpose or peroxide solutions for soft contact lenses can be used. The solution must be fresh and replaced every day. If contact lenses are not worn for several days, they must be disinfected before use.
- ELIMINATION OF PROTEIN DEPOSITS:** Use a protein removal product, or a multipurpose solution containing a protein removal agent. It is recommended that the lens storage case is changed every 90 days.
- Tap water, saliva, or products intended for gas permeable contact lenses should not be used to rinse or clean soft contact lenses.
- Always adhere to expiration dates of contact lens care solutions and adhere to the recommendations in the manufacturer's product information.
- To rinse the case of contact lenses with multipurpose solution and dry at air, at most one time per week.
- Proper contact lens care greatly reduces the risk of all contact lens-related eye infections, including those caused by *Acanthamoeba*.
- 5. STORAGE CONDITIONS**
- Avoid extreme temperatures: do not leave lenses near heat sources or in the sun, in the glove compartment of your car on warm days, or in a refrigerator or freezer.
- See contact lens labelling for the specific range of storage temperature.
- Renew the maintenance solution in the lens storage case every time you wear the lenses to ensure proper disinfection.
- Scan the QR code to obtain the recycling tips and the meaning of the symbols.
- 6. CONTRAINDICATIONS**
- Corneal hypoaesthesia.
- Eye or eyelid inflammation or infection.
- Induced or exacerbated allergic reactions due to contact lens wear or contact lens solutions use.
- Any eye disease, injury or abnormality that affects the cornea, conjunctiva or eyelids (recurrent corneal erosions, epithelial fragility, severe dry eye, red or irritated eyes, etc.).
- Any previously diagnosed condition that makes contact lens wear uncomfortable.
- Any systemic disease that may affect the eye or be exacerbated by wearing contact lenses.

7. PRECAUTIONS

1. Do not use the lenses if the original seal is broken. 2. Always disinfect the lenses before use. 3. Do not change the maintenance system, lens type, or lens parameters without first consulting your practitioner. 4. If a lens accidentally dries out, dispose of it. 5. Eyes should always be closed when applying hair spray as it may damage your lenses. 6. Make-up can discolour the lenses, so it is recommended to put your lenses in before applying your make-up. 7. Lenses should not be used in swimming pools, lakes and beaches except with goggles. If for any reason lenses come into direct contact with the water, wait 30 min before removing them and disinfect them immediately. Contact with unsterile water (e.g., from the tap, bottled, bath, pool, or water sports) increases the risk of microbial infection. 8. Do not use lenses in environments exposed to chlorine gas or other vapours or liquids. 9. Consult your eye care practitioner about using lenses while performing activities that could be considered dangerous. 10. Avoid any contact between the lenses and soap or shampoo as the eyes could become irritated. 11. Adhere to the regular eye check-ups recommended by your practitioner. 12. To guarantee optimum health for your eyes, renew your lenses regularly in accordance with the instructions included in the packaging or the wearing schedule stipulated by your practitioner. 13. The lenses should not come into contact with any substances that could discolour them or affect your health. Carefully read the instructions for use or consult your eye care practitioner. 14. Although some lenses provide UV and blue light protection, they are not sunscreen substitutes for sunglasses. 15. Do not share lenses with anyone. 16. Do not wear lenses while sleeping. 17. Always keep spare lenses or back-up spectacles with you.

8. UNDESIRABLE SIDE-EFFECTS

The worst scenarios for reportable incidents are: permanent deterioration of vision (loss of structure or function) and/or damage in eye that requires a medical treatment which leaves lasting effects.

In general, the potential undesirable side-effects when fitting contact lenses may happen mainly on the cornea and the conjunctiva that appear as a fitting complication:

1. Inflammation
2. Infection
3. Hypoxia
4. Mechanical issues
5. Allergy exacerbated by contact lens wear
6. Dryness

In the event of a serious incident relating to the product, it should be reported to the manufacturer and the competent authority of the Member State in which the user is established.

9. WARNING

Do not use your lenses if they are damaged or their expiry date exceeded.

Contact lens cases can be a source of bacterial growth.

Not following the wearing guidelines given by the eye care professional can lead to loss of vision and serious injury to the eye.

The risk of ulcerative keratitis among contact lens users who smoke is greater than among non-smokers.

Certain medications may cause blurred vision, lens dryness, or lens discolour (antihistamines, decongestants, anticholinergics, muscle relaxants, tranquilizers, and motion sickness medications).

If chemicals of any kind (household products, gardening solutions, laboratory chemicals, etc.) are splashed into the eyes, FLUSH THE EYES IMMEDIATELY WITH TAP WATER AND IMMEDIATELY CONTACT THE EYE CARE PRACTITIONER. If you experience any of the following symptoms: Irritation, discomfort, pain, red eyes, foreign body sensation, excessive tearing, abnormal secretions, major changes in vision or halos around lights.

REMOVING YOUR LENSES IMMEDIATELY AND CONSULT YOUR EYE CARE PRACTITIONER. If you use lenses while pregnant, you may experience symptoms of dryness and slight changes in refraction. Consult your vision specialist if this happens. Lenses can be used by disabled, handicapped and illiterate patients. Any special training needed (to insert, remove or care for their contact lenses) will be provided by the eye care practitioner, who is responsible for determining a patient's capacity for contact lens wear.

Children (0 to 6 years) will require an adult to insert, remove and care for their lenses.

10. WEARING SCHEDULE RECOMMENDED BY YOUR PRACTITIONER

DAILY WEAR (HOURS):

REPLACEMENT: ☐ Annual ☐ 3-Monthly ☐ Monthly ☐ Weekly

11. WASTE DISPOSAL

Scan the QR to see the Recycling tips. Contact lenses shall be disposed of in waste bins after use and never be disposed of in drains or toilets.

Boxes should be thrown into the cardboard container. Blisters and vials should be thrown into the plastic container.

Referencias.

1.Sankaridurg, P. et al. Myopia control with novel central and peripheral plus contact lenses and extended depth of focus contact lenses: 2 year results from a randomised clinical trial. *Ophthalmic Physiol. Opt.* 39, 294–307 (2019).

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