


PRODUCT CATALOGUE

Inspire your eyes
mark'envy

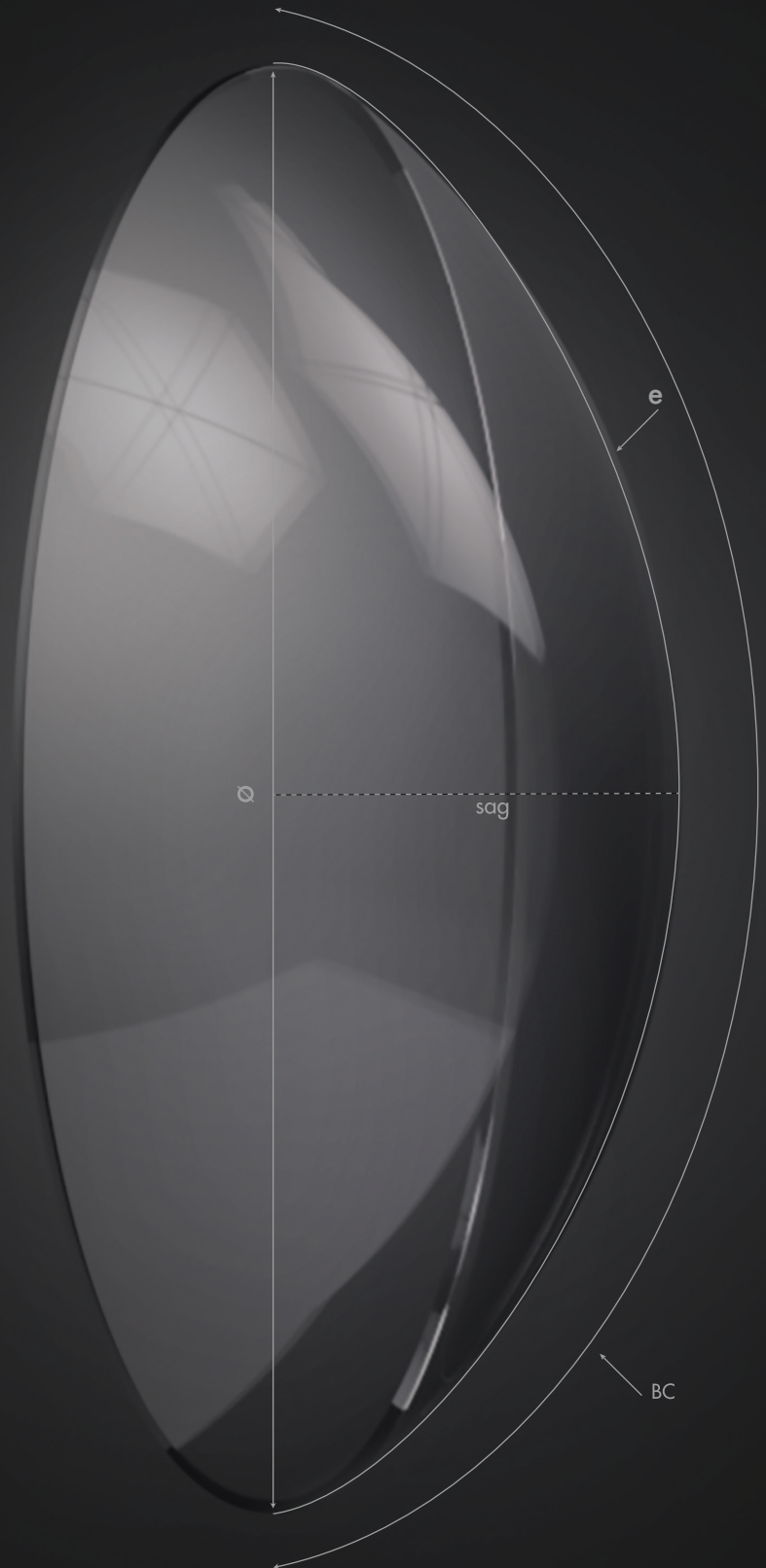

CUSTOM CONTACT LENSES FOR THE UNIQUENESS OF EACH EYE


- 241 SPHERES
- 180 AXES
- 30 CYLINDERS
- 15 ADDITIONS
- 11 DIAMETERS
- 4 GEOMETRIES
- 4 PROPRIETARY MATERIALS

Contact your business account manager to discuss the best options for increasing the number of patients you fit with mark'envy lenses

INDIVIDUALLY CRAFTED CONTACT LENSES FOR THE UNIQUENESS OF EACH EYE

MONTHLY SOFT CONTACT LENSES

Blu:gen | monthly

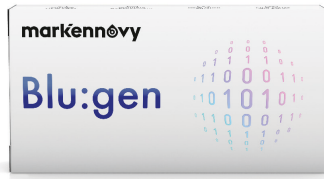

custom soft contact lens | 3 & 6-pack

Blu:gen is a silicone hydrogel lens, combining a Class 1 UV Filter with selective Blue Light Filtering to protect the eye from 99% of UVB, 93% of UVA, and 14% of harmful blue-violet light. Its high water content, low dehydration material featuring the lowest modulus of all silicone hydrogels on the market (0.25 MPa) offers your patients a healthy, comfortable all-day wearing experience.

Silicone Hydrogel (75% H₂O)
Blue Light Filter
Class 1 UV Filter

| | |
|---------------|---------|
| Dk | 60 |
| MPa | 0.25 |
| Handling Tint | Yes |
| Dehydration | < 0.50% |

| | |
|-------------------------------|---|
| <input type="radio"/> Spheric | <input checked="" type="radio"/> Multifocal |
| <input type="radio"/> Toric | <input checked="" type="radio"/> Multifocal Toric |

| | |
|-----|-----------------------------|
| BC | 6.50 to 9.80 (0.30) |
| Ø | 11.50 to 16.50 (0.50) |
| SPH | ±30.00 (0.25) |
| CYL | -0.75 to -8.00 (0.25) |
| AX | All (1°) |
| ADD | 0.50 to 4.00 (0.25) CD - CN |

Saphir Rx | monthly



custom soft contact lens | 3 & 6-pack

Saphir Rx is a silicone hydrogel lens, featuring a comfortable high water content, low dehydration material with a highly lubricious surface (CoF = 0.09). Its low modulus (0.29 MPa) adds to the comfort of the lens whilst ensuring vision quality and easy handling throughout the lens' life cycle.

Silicone Hydrogel (75% H₂O)

| | |
|---------------|------|
| Dk | 60 |
| MPa | 0.29 |
| Handling Tint | No |
| Dehydration | 1% |

| | |
|-------------------------------|---|
| <input type="radio"/> Spheric | <input checked="" type="radio"/> Multifocal |
| <input type="radio"/> Toric | <input checked="" type="radio"/> Multifocal Toric |

| | |
|-----|-----------------------------|
| BC | 6.80 to 9.80 (0.30) |
| Ø | 13.00 to 16.00 (0.50) |
| SPH | ±30.00 (0.25) |
| CYL | -0.75 to -8.00 (0.25) |
| AX | All (1°) |
| ADD | 0.50 to 4.00 (0.50) CD - CN |

Gentle 80 | monthly



custom soft contact lens | 3 & 6-pack

Gentle 80 is a bio-inspired hydrogel lens designed to imitate the natural properties of the cornea. Its material combines high water content, low dehydration, and the lowest modulus on the market (0.13 MPa) with oxygen transmissibility that reaches silicone hydrogel levels (Dk = 60), achieving award-winning comfort and health.

Origen Technology (80% H₂O)
UV Blocking

| | |
|---------------|---------|
| Dk | 60 |
| MPa | 0.13 |
| Handling Tint | Yes |
| Dehydration | < 0.50% |

| | |
|---|---|
| <input checked="" type="radio"/> Aspheric | <input checked="" type="radio"/> Multifocal |
| <input type="radio"/> Toric | <input checked="" type="radio"/> Multifocal Toric |

| | |
|-----|-----------------------------|
| BC | 7.10 to 9.80 (0.30) |
| Ø | 13.00 to 16.00 (0.50) |
| SPH | ±30.00 (0.25) |
| CYL | -0.75 to -8.00 (0.25) |
| AX | All (1°) |
| ADD | 0.50 to 4.00 (0.50) CD - CN |

INDIVIDUALLY CRAFTED

Our individually crafted soft lenses offer our broadest range of options

241

Spheres

180

Axes

30

Cylinders

15

Additions

12

Base Curves

11

Diameters

4

Geometries

4

Proprietary Materials

Inspire your eyes

markennovy

MONTHLY SOFT CONTACT LENSES

Gentle 59 | monthly



custom soft contact lens | 3 & 6-pack

Gentle 59 is a bio-inspired hydrogel lens designed to imitate the natural properties of the cornea. It combines high surface lubricity (CoF = 0.05) with low dehydration (< 1%) for excellent comfort. Its modulus (0.36 MPa) has been carefully calibrated to achieve optimal handling and vision quality throughout the lens' lifecycle, without reducing comfort or health.

| | |
|--|-----------------------------|
| Ori:gen Technology (59% H₂O) | |
| UV Blocking | |
| Dk | 30 |
| MPa | 0.36 |
| Handling Tint | Yes |
| Dehydration | < 1% |
| ☉ Spheric | ☉ Multifocal |
| ○ Toric | ⊗ Multifocal Toric |
| BC | 7.10 to 9.80 (0.30) |
| ∅ | 13.00 to 16.00 (0.50) |
| SPH | ±30.00 (0.25) |
| CYL | -0.75 to -8.00 (0.25) |
| AX | All (1°) |
| ADD | 0.50 to 4.00 (0.50) CD - CN |

Blu:kidz | monthly



custom soft contact lens | 3 & 6-pack

Blu:kidz is a silicone hydrogel lens, combining a Class 1 UV Filter with selective Blue Light Filtering to protect the eye from 99% of UVB, 93% of UVA, and 14% of harmful blue-violet light. Its child-friendly range of diameters makes it possible to fit even the smallest of eyes, whilst its green handling tint and high water content, low dehydration material provide improved handling and comfort – perfect for first-time contact lens wearers!

| | |
|---|-----------------------------|
| Silicone Hydrogel (75% H₂O) | |
| Blue Light Filter | |
| Class 1 UV Filter | |
| Dk | 60 |
| MPa | 0.25 |
| Handling Tint | Yes |
| Dehydration | < 0.50% |
| ☉ Spheric | ☉ Multifocal |
| ○ Toric | ⊗ Multifocal Toric |
| BC | 6.50 to 9.80 (0.30) |
| ∅ | 11.50 to 16.50 (0.50) |
| SPH | ±30.00 (0.25) |
| CYL | -0.75 to -8.00 (0.25) |
| AX | All (1°) |
| ADD | 0.50 to 4.00 (0.25) CD - CN |

Collect the Necessary Biometric Data

- Spectacle Prescription (up-to-date)
- Dominant Eye
- HVID
- K-Readings



Need fitting advice?

Talk to one of our experts:

T. 0800 328 0610

CALCULATE LENS ∅ & BC

Fitting Guide for Gentle 80

Lens ∅: Add 3mm to HVID | BC: Km = (K1 + K2)/2

| L∅ (mm) | BC (mm) | Km=(K1 + K2)/2 |
|---------|-------------|----------------|
| 13.00 | 7.10 - 8.90 | Km +0.0 |
| 13.50 | 7.10 - 9.20 | Km +0.0 |
| 14.00 | 7.40 - 9.50 | Km +0.1 |
| 14.50 | 7.70 - 9.80 | Km +0.3 |
| 15.00 | 8.00 - 9.80 | Km +0.5 |
| 15.50 | 8.30 - 9.80 | Km +0.7 |
| 16.00 | 8.60 - 9.80 | Km +0.9 |

Fitting Guide for Gentle 59

Lens ∅: Add 3mm to HVID | BC: Km = (K1 + K2)/2

| L∅ (mm) | BC (mm) | Km=(K1 + K2)/2 |
|---------|-------------|----------------|
| 13.00 | 7.10 - 8.90 | Km +0.0 |
| 13.50 | 7.10 - 9.20 | Km +0.2 |
| 14.00 | 7.40 - 9.50 | Km +0.4 |
| 14.50 | 7.70 - 9.80 | Km +0.6 |
| 15.00 | 8.00 - 9.80 | Km +0.8 |
| 15.50 | 8.30 - 9.80 | Km +1.0 |
| 16.00 | 8.60 - 9.80 | Km +1.2 |

Fitting Guide for Saphir RX

Lens ∅: Add 3mm to HVID | BC: Km = (K1 + K2)/2

| L∅ (mm) | RB (mm) | Km=(K1 + K2)/2 |
|---------|-------------|----------------|
| 13.00 | 6.50 - 8.90 | Km +0.0 |
| 13.50 | 7.10 - 9.20 | Km +0.1 |
| 14.00 | 7.40 - 9.50 | Km +0.3 |
| 14.50 | 7.70 - 9.80 | Km +0.5 |
| 15.00 | 8.00 - 9.80 | Km +0.7 |
| 15.50 | 8.30 - 9.80 | Km +0.9 |
| 16.00 | 8.60 - 9.80 | Km +1.1 |

Fitting Guide for Blu:gen, Blu:kidz

Lens ∅: Add 3mm to HVID | BC: Km = (K1 + K2)/2

| L∅ (mm) | RB (mm) | Km=(K1 + K2)/2 |
|-------------|-------------|----------------|
| 11.50/12.00 | 6.50 - 8.30 | Km +0.0 |
| 12.50 | 6.50 - 8.60 | Km +0.0 |
| 13.00 | 6.50 - 8.90 | Km +0.0 |
| 13.50 | 6.80 - 9.20 | Km +0.1 |
| 14.00 | 7.10 - 9.50 | Km +0.4 |
| 14.50 | 7.40 - 9.80 | Km +0.5 |
| 15.00 | 7.70 - 9.80 | Km +0.7 |
| 15.50 | 8.00 - 9.80 | Km +0.9 |
| 16.00 | 8.30 - 9.80 | Km +1.1 |
| 16.50 | 8.60 - 9.80 | Km +1.3 |

At mark'envoy, we are committed to challenging the contact lens industry status quo through innovation with a professional team that rivals any. The patient and eye care professional are firmly at the heart of everything we do. And we are passionate about adding value at every stage of the contact lens journey, from a patient's first consultation to the final fit.

The contact lenses we produce are carefully and individually crafted to the specific measurements and requirements of the patient's eyes as assessed by the eye care professional. Our most advanced materials, optics and technology enable us to provide exceptional, comfortable vision throughout the day, whilst safeguarding the health of patients' eyes. After all, at mark'envoy, we strongly believe that seeing better is living better!


TALK TO OUR CUSTOMER CARE TEAM:

T. 0800 328 0610 F. 0800 328 0649
E. mkservices@markenvoy.com
www.markenvoy.com



EXCLUSIVE FOCUS ON YOU

we only sell through you, the EYE CARE PROFESSIONAL

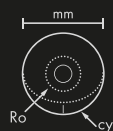

CUSTOM-MADE SOFT LENSES

we offer an exceptional COMBINATION OF PARAMETERS, GEOMETRIES AND LATEST-GENERATION MATERIALS so you can fit virtually any patient



TECHNICAL EXPERTISE

our customer care is STAFFED BY OPTICIANS, because we believe optician-led support adds greater value to you



FAST DELIVERY

we typically SHIP IN ONLY 72 HOURS, meaning you have custom lenses back in practice 4-5 WORKING DAYS AFTER PLACING YOUR ORDER



INDIVIDUALLY YOURS

every contact lens is carefully inspected and placed in the final package, engraved with your patient's name