


INDIVIDUALLY CRAFTED

Our individually crafted soft lenses offer our broadest range of options



MYLO is an individually crafted silicone hydrogel contact lens specifically designed for Myopia Management. It is powered by the **Brien Holden Vision Institute's** patented **Extended Depth of Focus (EDOF)** technology, which slows myopia progression and supports a comfortable acclimation to the lens, enhancing the overall wearing experience. A monthly disposable contact lens, **MYLO** features high water content and low coefficient of friction, which combine to improve comfort throughout the day. Its wide range of parameters and low elastic modulus ensure an excellent fit and easy handling, especially for the youngest contact lens wearers.

REPLACEMENT	Monthly	
PACK SIZE	3 & 6 lenses	
GEOMETRIES	☉ Spheric	
MATERIAL CHARACTERISTICS	Silicone Hydrogel	
	75% H ₂ O	
	Dk	60
	Dk/t	75
	MPa	0.29
	CoF	0.09
PARAMETERS	Ct	0.08
	BC (mm)	7.10 to 9.80 (0.30)
	Ø (mm)	13.50 to 15.50 (0.50)
	SPH (D)	-0.25 to -15.00 (0.25)

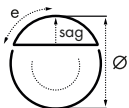

NEED FITTING ADVICE? TALK TO ONE OF OUR EXPERTS

Customer Care: 0800 328 0610

Inspire your eyes

markennovy

STEP-BY-STEP FITTING GUIDE FOR MYOPIA MANAGEMENT LENS

Do not forget to collect the biometric data of patients:

- HVID
- Spectacle Prescription (up-to-date)
- K-readings
- Eccentricity

1. Calculate the lens: to calculate the most precise diameter and base curve, please visit the Online Fitting Calculator (markennovy.com/fitting-calculator/)



2. Evaluate Fitting: check if diameter and base curve are correct

3. Evaluate monocular and binocular distance Visual Acuity (VA): monocular VA could be a slightly reduced compared to spectacles, i.e., less than a line. Binocular VA will be higher than or very close to 20/20 (1.0).

- If monocular VA is lower than 16/20 (0.8) monocularly, negative over-refraction will be applied: -0.25D should increase one line of VA.
- If not, keep the same power.


Contact your Business Development Manager for more details

Customer Care: T. 0800 328 0610 F. 0800 328 0649
International Customer Care: T. +34 916 496 123
mkservices@markennovy.com

OUR DIFFERENCE



Exclusive focus on the Eye Care Professional

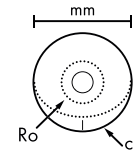


Exceptional combination of parameters, geometries and materials



Optician-led Customer Care



Individually Yours

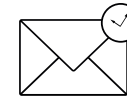

Fast delivery – diagnostic lenses shipped in 72 hours